
















KEY DESCRIPTIONS (#) Denotes item in Figure 1

-  (21) Make/Answer/End a call
Activates the speakerphone if pressed during a call
-  (22) Last number redial or insert a pause when pre-dialing a number
-  (6) Up or Down or scroll for menu options. Enter the CID book
-  Soft Keys (4 & 5) Perform the function indicated by the text immediately over it (on the bottom line of the display).
-  (23) Press to add an extra level of amplification & tone
-  (14) Voice Mail phone number
-  (24) Emergency phone number
-  (25) Use to store numbers
-  (15) To disconnect a call and re-establish dial tone or to switch over to another caller if you use call waiting

ANSWERING MACHINE KEY DESCRIPTIONS

-  Press to start or stop the Message or Memo playing
-  Press to play the next message
-  Press once to play the current message
Press twice to play the previous message
-  When playing an incoming message, press this key to delete the current message
-  Press and hold this key to record a memo (it will be stored as a new message)
-  Use to adjust the volume of the ringer, talking dialing and talking Caller ID

MENU OPERATION

To Access a MENU Option

1. To display the menu list, press the **Menu** soft key.
2. To scroll through menu options, press **▲** or **▼** repeatedly.
3. To select a menu item, Press the **Select** soft key when the desired menu item appears on the display.
4. Repeat if necessary.

To Exit from any MENU

If you press the **Back** soft key or the **Exit** soft key, the phone returns to the previous screen.

MENU MAP

Set Display

Set Language (N/A)
Set Contrast
Level 1-7
Dim: X--->X
ON or OFF
Hour Format
12-hr clock or 24-hr clock
Date Format
MM/DD or DD/MM

Set Phone

Set Ring
Day volume
Night volume
Ringer tone
VIP tone
Keypad Tones
Call Waiting
ON or OFF

Set Flash
100ms, 110 ms, 300ms, 600ms
Dialing Mode
Tone or Pulse

Date and Time

Set Talk

DIALING TALKING
ON or OFF
CID TALKING
ON or OFF
REVIEW TALKING
ON or OFF

TAM setup

TAM ON/OFF
Outgoing Msg
Play OGM
Record OGM
Change OGM
Erase OGM
Ring Number
3 to 6 rings, toll saver
Security ID
ICM Voice Slow

TELEPHONE ANSWERING MACHINE (TAM) OPERATION

To Turn the Answering Machine ON/OFF

1. Press the **Menu** soft key to display the main menu list.
2. Press **▲** or **▼** to move to **TAM setup** and then press the **Select** soft key.
3. Press the **Select** soft key to choose **TAM ON/OFF**.
4. Press **▲** or **▼** to **ON** or **OFF** and then press the **Save** soft key.
5. To exit, press the **Back** soft key to return to the previous screen and then press the **Exit** soft key to return to the main screen. Otherwise, after 1 minute, the display will automatically return to the main screen.



ClearSounds®
Hear how good life can be



QUICK START GUIDE

ClearSounds® CSC1000
Amplified Freedom Phone™
with Full ClearDigital Power™

Service in United States and Canada, contact:

ClearSounds Communications
Attention: Repair Center
1743 Quincy Avenue, Suite 155
Naperville, IL 60540 USA

800-965-9043 or www.clearsounds.com


Printed manual available upon request

PARTS CHECKLIST

PHONE BASE



HANDSET & HANDSET CORD

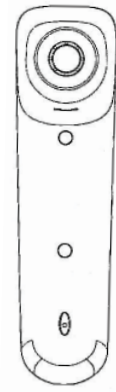


TELEPHONE LINE CORDS (ONE 8 INCH & ONE 6 FOOT CORD)



AC ADAPTER (7.5v/500mA)



WALL PLATE ADAPTER

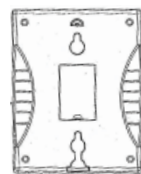

USER'S MANUAL



PHOTO TEMPLATE



CONNECTING YOUR PHONE

1. CHECK PARTS LIST

2. PRE-SELECT AUXILIARY FEATURES:

- a. AMPLIFICATION: Amplify Override ON / OFF
- b. RINGER: Strobe & Shaker / Only Shaker / Only Strobe

3. PHONE INSTALLATION: The phone should be located in an area where it is easy to see the visual display and lighted ringer. Choose an area near a telephone wall jack (RJ11C) and power outlet. The telephone may be placed on a shelf or table or may be fixed to the wall using the wall plate adapter (mounting bracket). **Begin by removing the clear plastic Caller ID protective tab.**

Desk / Table Mounting

1. Plug the AC power adapter into the jack on the back of the telephone. Wait until the phone is completely setup before plugging the AC adapter into the electrical outlet or surge protector.
2. Connect one end of the 6 ft telephone cord to the line jack on the back of the phone and connect the other end to a modular phone jack.
3. Connect one end of the coiled handset cord to the handset and the other end to the phone base.
4. Lift the handset and listen for the dial tone.

Wall Mounting

1. Adjust the handset hanger tab so that the hook points up. This will keep the handset from falling out of the cradle when it is mounted on the wall.
2. Plug the AC power adapter into the jack on the back of the telephone (complete setup before plugging the AC adapter into the electrical outlet or surge protector).
3. Plug the 8-inch short phone cord into the telephone jack on the phone.
4. Insert the wall plate adapter by lining it up with the holes on the phone, then push the wall plate adapter in towards the phone and slide the plate upwards until it is securely seated on the phone.
5. With the wall plate adapter installed, plug the 8-inch short phone cord into the jack on the wall.
6. Mount phone onto wall jack mounting screws (dry wall mounting, see page 14).
7. Plug the AC power adapter into the electrical outlet.
8. Connect the handset using the coiled handset cord and listen for a dial tone.

LOCATION OF CONTROLS

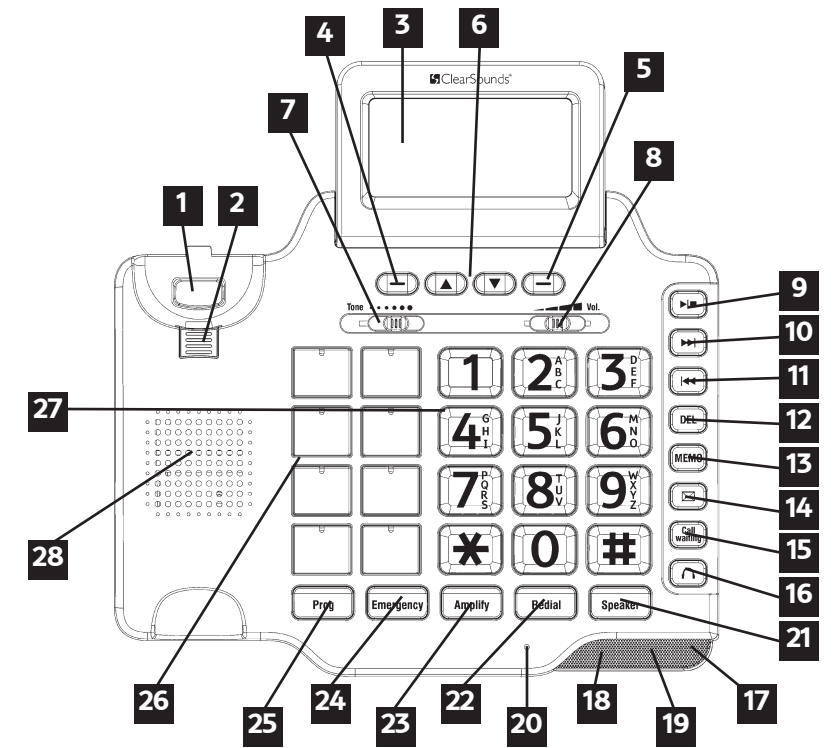


Figure 1

- | | |
|--------------------------------|--|
| 01. Hook Switch | 16. Headset Key |
| 02. Handset Hanger Tab | 17. Message waiting / New call LED |
| 03. Large LCD/Caller ID Screen | 18. In Use LED Indicator |
| 04. Left Soft Key | 19. Visual Phone Ringer |
| 05. Right Soft Key | 20. Microphone |
| 06. Up/Down Scroll | 21. Speakerphone Key |
| 07. Handset Tone Control | 22. Redial Key |
| 08. Handset Volume Control | 23. Amplify Activation Key |
| 09. Play/Stop Key | 24. Emergency Memory Key |
| 10. Next Key | 25. Prog – Program Key |
| 11. Previous Key | 26. (8) 1-Touch Photo Memory Dial Keys |
| 12. Delete Key | 27. Lighted Keypad |
| 13. MEMO Key | 28. Speaker |
| 14. Voice Mail Key | |
| 15. Call Waiting Key | |

If at any time you have questions about your telephone after reading this manual, please contact our ClearSounds® Customer Service Department at 1-800-965-9043.