

Setup Guide

1 Unpack



A500E DECT 6.0
Amplified Cordless
Expandable Handset



Please see the *ClearSounds A500 DECT 6.0 Amplified Cordless Phone with Answering Machine User's Guide* for additional instructions.



Telephone handset



A500E charger



AC power adapter assembly




Three rechargeable AAA
NiMH batteries





Carrying clip

2 Power

Plug in

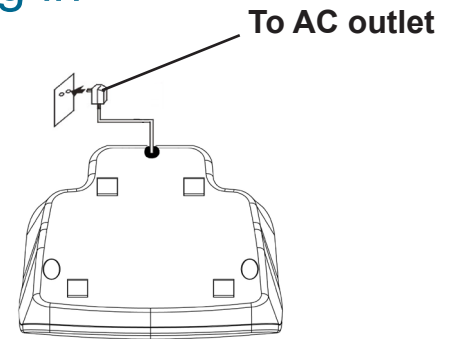

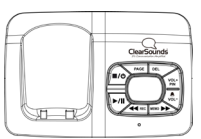



Charge

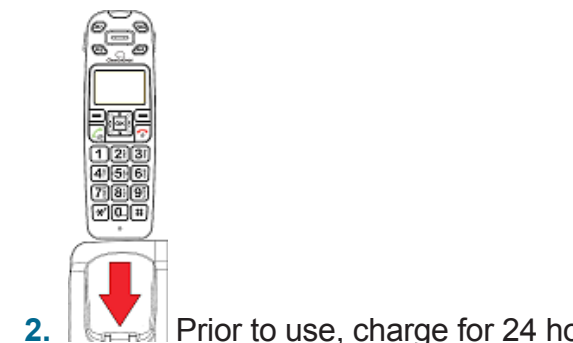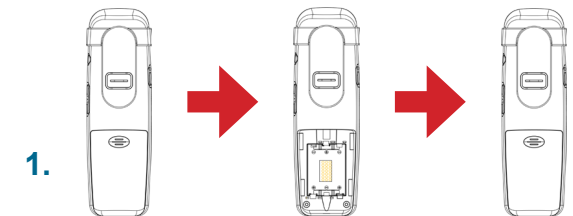



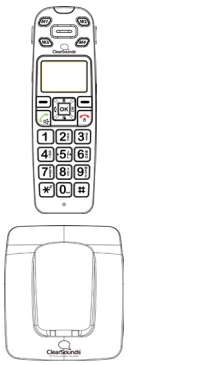


Prior to use, charge for 24 hours.





3 Register

	<p>1. On the unregistered A500E handset, press and hold  beneath Menu on the screen, until Press and hold base Page key appears, and then release the handset key.</p>
	<p>2. On the answering machine base, press and hold  for 10 seconds, and then release the key.</p> <p>When the handset is registered to the base, the handset number appears on the handset screen.</p>

4 Call

Place	Answer	End
<p>1. Dial .</p> <p>2. Press .</p>	<p>1.  Lift.</p> <p>2. Press .</p>	<p>1. Press .</p>

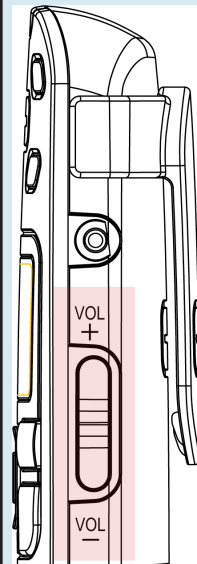5 Volume

Volume	Amplify	Equalize
 <ul style="list-style-type: none"> • Press + or -. 	 <ul style="list-style-type: none"> • Press to turn on or off. 	<p>1. Press .</p> <p>2. Press  to select Treble, Flat or Bass.</p> <p>As you select, a graph appears on the screen.</p>

ClearSounds 1-800-965-9043