

## KEY DESCRIPTIONS

### Handset



#### Dial Key

Make/Answer a call  
Activates the speakerphone if pressed during a call



#### End Key

End a call  
Long press to activate power on/off



#### Navigation Key:



Up scroll for lists and menu options  
Increase the earpiece/speakerphone volume during a call  
Enter the Caller ID (CID) book



Down scroll for lists and menu options  
Decrease the earpiece/speakerphone volume during a call  
Enter the Phonebook



Set up an internal call to another handset  
Transfer an incoming call to another handset  
Set up a conference call with an external line and an internal handset



Enter the redial list  
Insert a pause when pre-dialing a number or storing a phonebook entry



**Soft Keys** – Two soft keys are used to select functions displayed directly above each of the soft keys (on the bottom line of the display). Functions displayed above the soft key will change as you move through the menu.



**Star Key** - Press and hold to turn on/off keypad lock function  
Set up a conference call



**One-touch memories** – Store up to 4 numbers for one-touch memory dialing



**O Key** – Press and hold to activate the shaker and visual ringer function

## MENU MAP (p. #) denotes page in Operator Manual

Main Menu	Submenu I	Submenu II
CID (Caller ID) Book (p.41)	Add to PB (p.44) Delete (p.44) Delete All (p.44)	
Phonebook (PB) (p.57)	New (p.58) Edit (p.60) Delete (p.61) Delete All (p.61) PB Status (p.61)	
BS (Base) Settings (p.26)	Terminate HS (p.29) Dial Mode (p.27) Flash Time (p.27) Modify PIN (Personal Identification Number) (p.28) BS Reset (p.29)	
HS (Handset) Settings (p.30)	Audio Setup (p.32)	Earpiece V. (vol.) (p.32)
	Ring Setup (p.22)	INT (Internal) Ring (p.23) EXT (External) Ring (p.23)
	Tone Setup (p.34)	Key Tone (p.34) Range Alarm (p.31)
	Language (p.37) HS Name (p.30) Auto Answer (p.32) LCD Contrast (p.38)	
	Date & Time (p.39)	Date Format (p.39) Time Format (p.39) Set Date (p.39) Set Time (p.39)
	Select Base (p.31) HS Reset (p.30) Amplified (p.21) Talk Keypad (p.34) Talk CID (p.33)	
	Registration (p.19)	

## TELEPHONE ANSWERING MACHINE MENU MAP

Main Menu	Submenu I	Submenu II
Announce (p.49)		
Message (p.52)	New MSG (p.53) Saved MSG (p.53) Delete All (p.54) Memo Record (p.55)	
TAM Setting (p.47)	TAM On/Off (p.46) TAM Alert (p.48) ANNC Select (p.48) Set Ring (p.47) TAM Language (p.47) TAM Default (p.57)	



ClearSounds®  
Hear how good life can be



## QUICK START GUIDE

### ClearSounds® A600E\* DECT 6.0 Expandable Handset


\* Requires use of the A600 Amplified Freedom™  
Phone with Full ClearDigital™ Power

**Service in United States and Canada, contact:**  
ClearSounds Communications  
Attention: Repair Center  
1743 Quincy Avenue, Suite 155  
Naperville, IL 60540 USA

**800-965-9043 or [www.clearsounds.com](http://www.clearsounds.com)**  
Printed manual available upon request

## PARTS CHECKLIST

### PHONE BASE

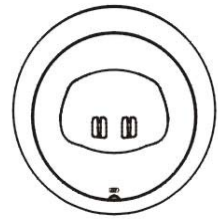

### CORDLESS HANDSET



### BATTERIES



### CARRYING CLIP



### AC ADAPTER








### USER'S MANUAL



## REGISTERING ADDITIONAL HANDSETS

The supplied handset is already registered to the base unit, and you can register up to 4 additional handsets to the base.

1. Before registering an additional handset to the base, you should press and hold the page key , which is located on the main base, for about 5 seconds and then release it.
2. Press the **Menu** soft key on the handset you are registering to display the main menu list.
3. Press up  or down  to scroll to **Registration** and then press the **Select** soft key.
4. Press up  or down  to scroll through the base list and select one of the bases and then press the **Select** soft key. The handset will search for the requested base.
5. If the handset is successfully registered, you will be prompted to enter the PIN code (the default PIN is 0000). Upon validation of the PIN, the Registration tone will sound and the base assigns a number to the handset.

**More information on this feature located in the Operating Manual on page 19.**

## CONNECTING YOUR PHONE

### 1. CHECK PARTS LIST

**2. PHONE INSTALLATION:** The phone should be located in an area where it is easy to see the visual display and lighted ringer. **Begin by removing the clear plastic Caller ID protective tab. This film tab is only for display and to protect your Caller ID display during shipping.**

### Desk / Table Mounting







Plug the AC power adapter into the jack on the back of the base, then plug the AC adapter into a standard electrical outlet or surge protector.

**NOTE:** The adapter is required for the phone to work.

## TALKING SETTING

### Caller ID

Your ClearSounds® A600E comes from the factory preset for Talking Caller ID Mode set to ON and will announce the caller's number between rings. To change this setting:

1. Press the **Menu** soft key to display the main menu list.
2. Press up  or down  to scroll to **HS Settings** and then press the **Select** soft key.
3. Press up  or down  to scroll to **Talk CID** and then press the **Select** soft key.
4. Press up  or down  to select **On** or **Off** to activate/deactivate the Talking Caller ID function and then press the **Save** soft key.


## MEMORY KEYS

### Dialing from the Memory M1 – M4 and SOS keys

Press one of the **M1 – M4** or **SOS** memory dial keys and the phone will automatically begin to dial (Page 30 of the Operating Manual describes how to program the memory dial keys).

**If at any time you have questions about your telephone after reading this manual, please contact our ClearSounds® Customer Service Department at 1-800-965-9043.**

## LOCATION OF CONTROLS

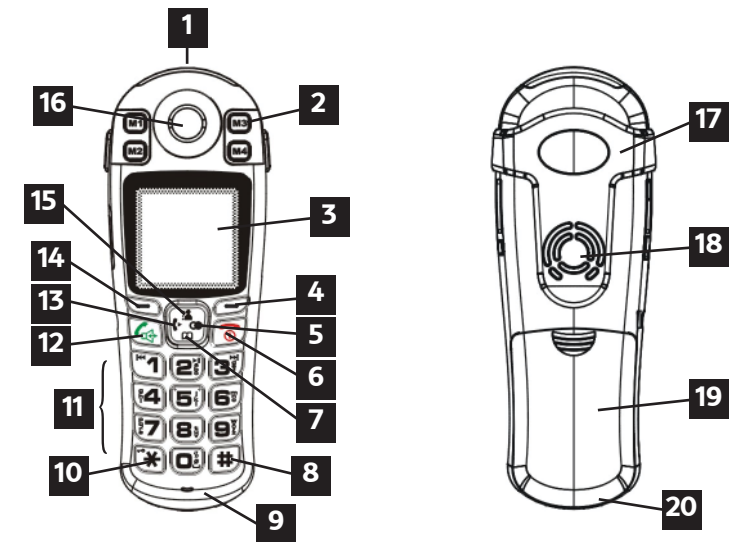




**Figure 1**


- |   |                              |
|---|------------------------------|
| 1 Visual Phone Ringer & Amplify Indicator Light | 11 Lighted Keypad            |
| 2 (4) 1-Touch Memory Dial Keys                  | 12 Dial/Speakerphone Key     |
| 3 Large LCD/Caller ID Screen                    | 13 Left/Intercom Key         |
| 4 Right Soft Key                                | 14 Left Soft Key             |
| 5 Right/Redial Key                              | 15 Up/Caller ID Key          |
| 6 End Key                                       | 16 Earpiece                  |
| 7 Down/Phonebook Key                            | 17 Carrying Clip             |
| 8 Hash (Pound) Key                              | 18 Speaker                   |
| 9 Microphone                                    | 19 Battery Compartment Cover |
| 10 Star Key                                     | 20 Charging Contacts         |

## TELEPHONE ANSWERING MACHINE (TAM) OPERATION

### **Method 1: Using the Handset**

1. Press the **TAM** soft key to display the TAM main menu list.
2. Press down  to scroll to **TAM Setting** and then press the **Select** soft key.
3. Press the **Select** soft key to select **TAM On/Off**.
4. Press up  or down  to select **Off** or **On** and then press the **Select** soft key.

### **Method 2: Using the Base**

1. With power coming to the base, press the  key on the base to turn the answering machine **Off** or **On**. The base will announce the selected mode.